

## bath board safety

- The bath board should only be used as an aid for sitting. Do not stand on or misuse the bath board.
- Do not exceed the weight limit of the bath board.
- Always ensure the bath board is secured in the bath before use.
- Ensure that the bath board does not overhang the edge of the bath by more than 2.5cm (1").
- Ensure that the correct width of bath board is used for each bath.
- Some users and children may require supervision whilst using the bath board.
- Keep hands and fingers clear of the gaps between the slats on the board.

## specification

The maximum user weight for the bath board is 150kg (23 stone)

The bath board measures 69cm (27") long and 31cm (12") wide.

It is suitable for baths with an inside measurement of 40cm - 63cm (16" - 25").

It is constructed from high quality PP plastic.

## warranty details

Your Drive branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive stockist.



### Drive Medical Ltd

Ainley's Industrial Estate, Elland,  
West Yorkshire, GB HX5 9JP  
[www.drivemedical.co.uk](http://www.drivemedical.co.uk)

DOC CODE: 12023\_IFU (Iss 11/13)

# bath board



[www.drivemedical.co.uk](http://www.drivemedical.co.uk)


## installation

The bath board contains:


1 x bath board

4 x adjustment cones with screw and washer


1 x handle / handgrip with screws

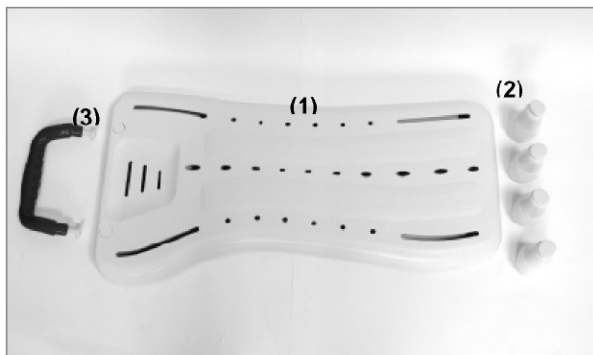


1. If necessary, remove the 2 plastic screws from the handle and remove the two plugs either side of the soap tray at the wide end of the board. Insert the handle and secure by screwing in the 2 plastic screws from the underside of the bench using a coin or similar.

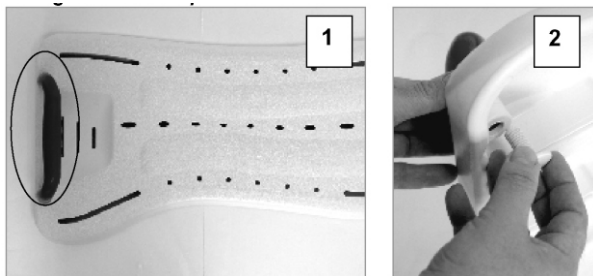

2. If necessary remove the plastic screws from the 4 cones supplied with the bath board. Insert the screws in to the slots on the bench, attach a washer to the under-side and then loosely screw on the cone.

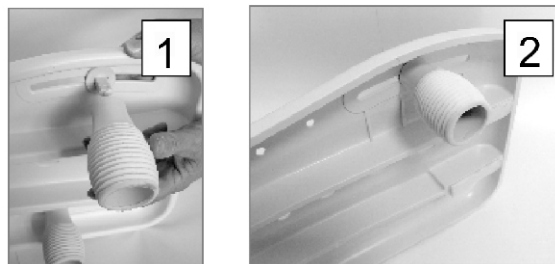

3. Slide the cones (as shown by photo) until they touch the inside walls of the bath. Remove the board, tighten the cones, and then reposition the board in the bath.

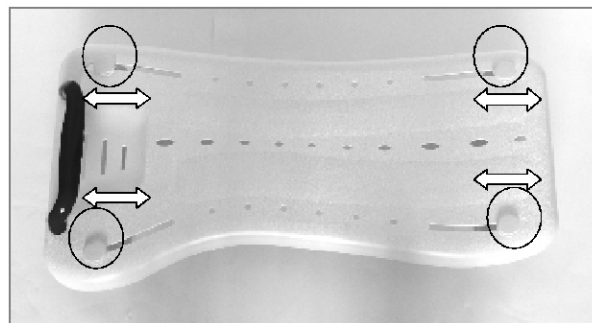

*Note: Photographs supplied for illustration purposes only.*

## using the bath board

Ensure you read the Safety section before using the bath board.

They are designed for a user to sit on in the bath, enabling them to bathe, wash or shower in comfort. They have a weight limit of 150kg (23 stone).

Consult your healthcare professional for advice about using the product safely and transferring to and from the bath.

## cleaning the bath board

Before being cleaned, the bath board should be removed from the bath and the brackets removed.

All parts of the bath board should be cleaned periodically using normal household detergents.

When disinfecting is required, a mild disinfectant that will not cause damage to plastics should only be used.

Strong acid, alkaline or cleaning products may cause damage or discolouration to the product.

After cleaning or disinfecting, the parts should be showered down or wiped with a damp cloth before the bath board is reinstalled.